

RESPIMAT®

Three simple steps for daily use: T.O.P.



Turn

- Keep the cap closed.
- **TURN** the clear base in the direction of the arrows on the label until it clicks (half a turn).



Open

- **OPEN** the cap until it snaps fully open.
- Breathe out slowly and fully, away from the inhaler.



Press


- Close your lips around the mouthpiece.
- While taking in a slow, deep breath, **PRESS** dose release button; keep breathing in slowly.
- Repeat Turn, Open, Press for a total of two puffs.

For demonstration videos and other resources, scan the QR code on your medication pack.

Keep your Respimat device.
Each device can be used with
up to **6 cartridges**.

**TWO
PUFFS
ONCE
DAILY**

HOW TO INSERT A NEW CARTRIDGE

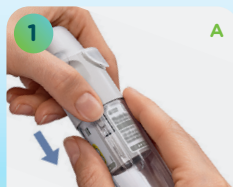

A



B



2



3



REMOVE CLEAR BASE

TO INSERT THE FIRST CARTRIDGE: **(A)**

- Press the safety catch while pulling off the clear base.

OR, TO REPLACE AN EMPTY CARTRIDGE: **(B)**

- When the dose indicator turns red with an arrow, turn the clear base until it detaches.
- Pull off the empty cartridge and dispose.

INSERT CARTRIDGE

- Insert the cartridge.
- Place the inhaler on a firm surface and push down until it clicks into place.

TRACK CARTRIDGE

- Mark the check-box on inhaler's label to track the number of cartridges.
- Push the clear base back into place until it clicks.



4

TURN

- Keep the cap closed.
- Turn the clear base in the direction of the arrows on the label until it clicks (half a turn).



5

OPEN

- Open the cap until it snaps fully open.



6

PRESS

- Point the inhaler toward the ground.
- Press the dose-release button.
- Close the cap.
- Repeat steps 4-6 until a cloud is visible.
- Then repeat steps 4-6 three more times.